

Thigh/Calf Sleeve

Instructions For Use

Indication:
For General
Support & Compression

Please read these instructions completely and carefully before applying the sleeve. Correct application of the sleeve is important for proper function.

Manufactured by:



TopShelf[®]
Manufacturing, LLC


1851 East Paradise Road, Suite A
Tracy, CA 95304

Phone: 866.592.0488 • Fax: 209.834.8832

APPLICATION AND FITTING INSTRUCTIONS


Thigh Sleeve

Grasp edges of the sleeve at the larger opening. Bring the sleeve onto the leg, pulling it onto the thigh to a comfortable level.

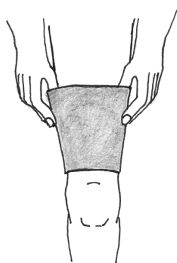

Calf Sleeve

Bring the sleeve onto the leg so that the calf is centered within the sleeve.

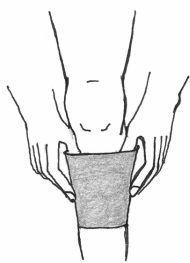

CARE INSTRUCTIONS

Hand wash in cool water utilizing mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this brace, consult your healthcare provider immediately.

Neoprene braces and supports should not be worn by individuals with known allergies to neoprene products or susceptibility to dermatitis.

WARRANTY

Top Shelf guarantees this product to be free of defects in material and construction for a period of six months from date of purchase.