

# Lateral Patella Stabilizer

## Instructions For Use

Indication:

For Patella Femoral  
(Knee Cap) Disorders

Please read these instructions completely  
and carefully before applying this brace.  
Correct application of the brace is  
important for proper function.

Manufactured by:




Tracy, CA 95304

Phone: 866.592.0488 • Fax: 209.834.8832

# APPLICATION AND FITTING INSTRUCTIONS

1. Grasp edges of the sleeve at the larger opening. Step through sleeve, bringing it on to the knee so that the opening at the front surrounds the knee cap (Patella).
2. Wrap and attach the bottom strap, then the top strap to the inside of the knee.

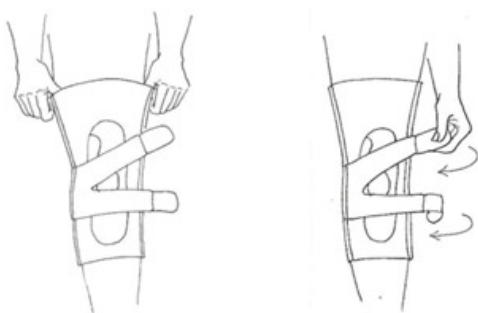

## CARE INSTRUCTIONS

Hand wash in cool water utilizing mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

## WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this brace, consult your healthcare provider immediately.

Neoprene braces and supports should not be worn by individuals with known allergies to neoprene products or susceptibility to dermatitis.

## WARRANTY

Top Shelf guarantees this product to be free of defects in material and construction for a period of six months from date of purchase.