

1851 East Paradise Road, Suite A
Tracy, CA 95304
Phone: 866.592.0488 Fax: 209.834.8832
topshelforthopedics.com



ReIOAd Knee Brace

Instructions For Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.



ReLOAD Knee Brace

Instructions For Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.





Indications: Unicompartmental knee osteoarthritis, articular cartilage defect repair, meniscal cartilage repair, osteochondral defect repair, and degenerative joint disease.

Application Instructions:

1. With the straps released and the thigh portion open - slide the brace onto the affected leg positioning the cutout over the knee cap.
2. Close the thigh cuff by overlapping the fabric to engage the hook and loop. Make sure knee cap is centered in the opening. (Attached patellar buttress on the inside of the knee cap opening can be rotated or removed to improve fit and/or function).
3. Secure the lower calf strap followed by the thigh strap by affixing the buckle assembly and adjusting the tension of the strap. (Buckles release by depressing the "wings" on both side of the device at the same time).
- 3a. The calf strap should be secured with the leg in full extension and the single upright metal strut pulled towards the back of the thigh and calf. Strap should be secured as close to the upper calf as possible to prevent migration of the brace.


Strap Adjustment: You can adjust the length of the straps if they are too long by removing the "Y" tab adapter at the end of the strap, cutting the strap to length and re-affix the "Y" tab piece (Inset image).

4. Adjust the telescoping dial hinge while standing by rotating it clockwise to the desired level of "pressure" to aid in the unloading of the opposite side of your affected knee joint. There are 4 separate positions to this component that provide up to an additional 6mm of unloading of your knee joint before it resets to zero (0).

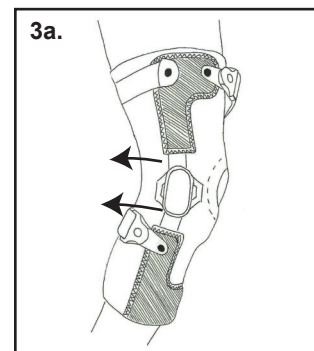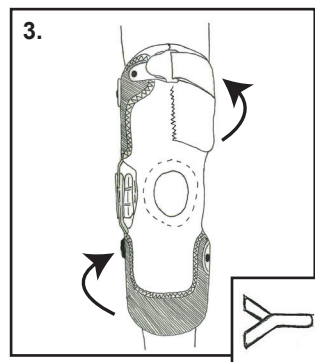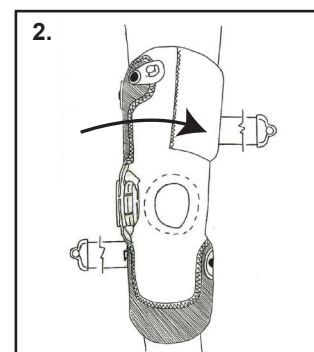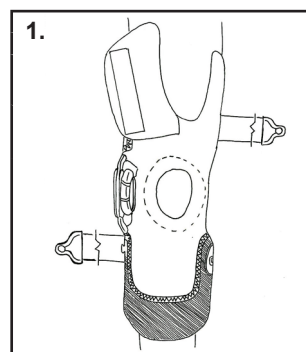

NOTE: If telescoping hinge is not touching your knee in full extension, adjust metal upright more posterior on the thigh and calf. Shaping the upright to match the contour of the thigh may be required for those with more than 15 degrees of varus or valgus of the knee.

Tele-scoping Hinge Dial Rotation

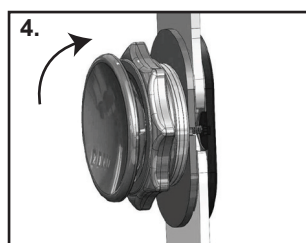

Use and Care:

Straps and pads may be trimmed to accommodate variations in leg size. Following activity in fresh or salt water, rinse brace thoroughly in clean water and air dry. Routinely check the hinge screws for tightness and retighten as needed. Lubricate hinges periodically with a dry lubricant or Teflon™ spray. Do not use petroleum based lubricants. Hand wash liners and condyle pads in cold water with a mild detergent. Air Dry, do not heat dry liners or pads.

Note:

Latex free.

Warning:

This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product consult your medical professional immediately

Notice:

While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through the use of this product.

Warranty:

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months on the associated soft goods and straps, and six months on the brace frame from the date of purchase.

Caution:

Federal Law (U.S.A.) restricts this device for sale by or on the order of a licensed health care professional.