

topshelforthopedics.com  
Phone: 866.592.0488 • Fax: 209.834.8832  
Tracy, CA 95304  
1851 East Paradise Road, Suite A



## Forefoot Offloading Shoe Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.



# Forefoot Offloading Shoe

## Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.

**Indications:** Stabilization of metatarsal fracture, post-op immobilization of phalangeal and metatarsal procedures.



1. Place foot inside the shoe and make sure the heel is touching the back of the shoe.
2. Secure the forefoot strap and ankle strap. Do not over tighten the straps.

### **Use and Care:**

Clean with damp cloth. Dry thoroughly before next use. Shoe may be hand laundered with mild detergent in warm water and air dried. The sole may be wiped clean with damp cloth.

### **Notes:**

For single patient use only.

### **Warning:**

This product is to be used under the supervision of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

### **Warranty:**

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from the date of purchase.

### **Caution:**

Federal Law (U.S.A.) restricts this device for sale by or on the order of a licensed healthcare professional.