

topshelforthopedics.com
Phone: 866.592.0488 • Fax: 209.834.8832
Tracy, CA 95304
1851 East Paradise Road, Suite A



BOA S.I. Belt

Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.








BOA S.I. Belt

Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.

Indications: Sacroiliac pain & dysfunction.

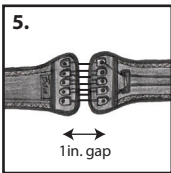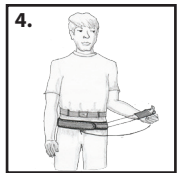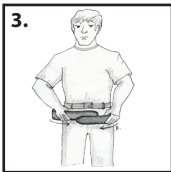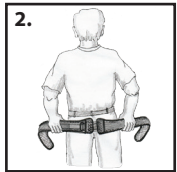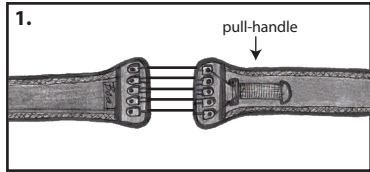

1. Position and fasten the pull-handle as close to the compression panels to fully extend the cord.
2. For proper fit and compression, center the two side panels and position them symmetrically on the back.
3. Position the brace approximately 1" above the greater trochanter (widest part of the hips) and fasten the belt in place by engaging the hook and loop on the distal portion of each panel. The pull-handle will be on your left side and the soft lining will be against the body.
4. To apply compression, grasp the pull-handle with the thumb and forefinger. After desired level of compression is achieved, fasten the pull-handle to the contact strip of loop on the side panel.
5. A properly tensioned assembly should have a 1" gap between the two posterior compression panels.

Use & Care: Hand wash in cold water using mild soap, rinse thoroughly. Air dry only.
Note: If not rinsed thoroughly, residual soap may cause skin irritation and deterioration of the material.

Warning: This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

Warranty: Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from date of purchase.

Caution: Federal Law (USA) restricts this device for sale by or on the order of a licensed healthcare professional.