

topshelforthopedics.com
Phone: 866.592.0488 • Fax: 209.834.8832
Tracy, CA 95304
1851 East Paradise Road, Suite A



ProteX Elbow Brace

Instructions For Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.




ProteX Elbow Brace

Instructions For Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.


Indications: Hyperextension of the elbow, UCL sprains, post elbow dislocation, chronic elbow injuries.

Application Instructions:

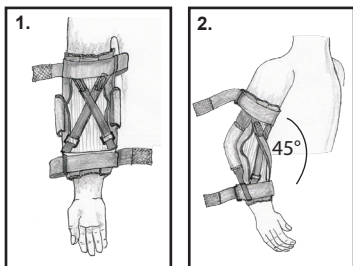


1. Loosen upper and lower straps to allow arm to enter brace. Insert arm into sleeve with the cross strapping oriented to the front of the arm and the smaller black "D" rings towards the wrist.
2. Orient the hinges at the bend of the arm by flexing (bending) the elbow 45 degrees.
3. Secure the upper and lower anchoring straps.
4. Adjust the anterior cross straps to provide additional protection from hyperextension as needed.

To insert an alternative extension stop:

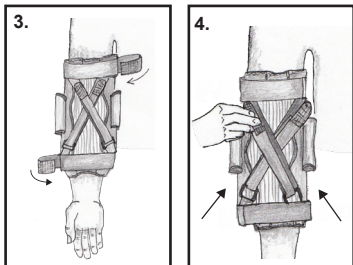

5. Expose the hinges on both sides by releasing the outer neoprene cover from the inside of the hinge cover.
 - a. Using a standard Phillips screwdriver, remove the lower set screw on both sides. Remove the current stops and replace them with the selected alternative 20, 30, 40 degree that is affixed to the plastic "trees" included with the brace - They are marked EXT for extension.

(Replace lower set screws. Re-affix neoprene cover on both sides).

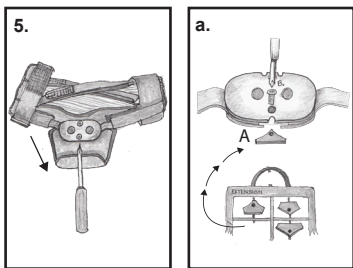

Use and Care:

Hand wash the brace with mild soap and water and lay out to dry. Do NOT machine dry.

Notes:

For single patient use only.

Warning:

This product is to be used under the supervision of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

Warning:

Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene.

Warranty:

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from the date of purchase.

Caution:

Federal Law (U.S.A.) restricts this device for sale by or on the order of a licensed healthcare professional.

